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This package insert contains important product use & safety information. Please read carefully.

INDICATIONS

When used as directed HORIEN ULTRA COMFORT Multipurpose Solution cleans, rinses, disinfects, lubricates, wets and stores soft contact lenses. HORIEN ULTRA COMFORT is also indicated for daily protein removal from lenses.

CONTENTS

A sterile, aqueous, isotonic solution containing Sodium Chloride, Potassium Chloride, Disodium Edetate, Polyhexanide, Poloxamer, Hypromellose and Sodium Phosphate buffer.

HORIEN ULTRA COMFORT is a power packed combination of four special agents, that cleans your lenses better, removes protein more effectively, makes your lenses germ free and keeps them moist so that they are more comfortable on your eye. pH & isotonicity are adjusted to lacrimal fluid.

Polyhexanide: A disinfectant & preservative with powerful biocidal efficacy. Keeps your lens free of germs & bacteria.

Poloxamer: A gentle surfactant & cleaning agent. Removes protein, dirt & debris.

Hypromellose: A lubricant that forms a shield on the lens. Retains moisture, prevents dryness & lubricates the lens for long-lasting eye comfort.

Sodium Phosphate Buffer: Keeps solution pH balanced & enhances performance of added electrolytes (potassium chloride & sodium chloride) for more natural lens wear.

DIRECTIONS FOR USE

- Always wash your hands and dry them with a clean towel before handling your lenses. Always start with the same eye to avoid confusion.
- Open the lens case.
- Remove lens and place in the palm of your hand.
- Apply at least 3 drops of HORIEN ULTRA COMFORT to each lens surface.
- Gently rub the lens using a backward and forward motion for 10 seconds per side.
- Rinse both lens surfaces thoroughly with HORIEN ULTRA COMFORT while rubbing the lens gently between the thumb and forefinger.
- Place the lens in the lens case and fill the well with HORIEN ULTRA COMFORT.
- Replace the cap and screw tight. Repeat the procedure with the other eye's lens.
- Allow your lenses to soak at least for 4 hours.
- Your lenses are now clean and disinfected and may be inserted directly into your eyes.
- Fast Clean Option – Rub & Soak for 4 Hours
- Rub Free Option – Soak for 6 Hours

HORIEN ULTRA COMFORT offers you a Rub Free cleaning option, in which case you need not rub the lens as instructed. However, you must rinse the lens thoroughly and soak it for at least six hours. This option is best used with frequent replacement lenses (1 month or less). Rubbing is the preferred cleaning method for conventional soft lenses.

INSERTING YOUR LENSES

- Make sure your hands are washed with a mild soap, rinsed thoroughly and dried with a lint-free towel before inserting your lenses.
- Remove the right eye's lens from the storage case. Clean & rinse lens with HORIEN ULTRA COMFORT.
- Place the lens on the tip of your right-hand index finger.
- Place the middle finger of the same hand close to your lower eyelashes and hold down the lower lid. Pull upper eyelid upwards with the middle finger of the left hand.
- Look straight ahead and gently place the lens on your eye.
- Gently release the lid and blink. The lens will center itself automatically.
- Repeat procedure on the left eye's lens.
- Use a mirror if necessary.
- Check to be sure that the lens is centered by covering the other eye. If your vision is crisp and clear then the lens is in place.

REMOVING YOUR LENSES

- Remove cosmetics, make-up only after you have removed the lens.
- Please ensure your hands are washed, rinsed and dried.
- Begin procedure with the right eye.
- Pull the lower lid down with the middle finger of your right hand.
- Look upward, slide lens to lower part of the eye with your forefinger.
- Gently pinch the lens between the thumb and forefinger and remove from the eye.
- Clean and rinse with HORIEN ULTRA COMFORT and put into the lens storage case before removing lens from the other eye.

CONTRA INDICATIONS (Reasons not to use)

There are no known contra indications for use of HORIEN ULTRA COMFORT, however if you are allergic to any ingredient in this solution, DO NOT USE.

HAVE A HEALTHY RELATIONSHIP WITH YOUR CONTACT LENS

- Avoid strong, oily soaps while washing your hands. Make sure you rinse and dry your hands before handling your lenses.
 - Make sure your surroundings are smooth & clean. If you are using a basin/sink, make sure the plug is in to ensure that the lens does not get drained away.
 - Always clean and disinfect lenses before re-insertion into the eye. Always use fresh HORIEN ULTRA COMFORT from the bottle. Never re-use solution left in the lens storage case.
 - Do not store or rinse lenses in tap water.
 - Use only the lens case provided to store your lenses.
 - Make sure the lens is fully immersed in the solution. Never use lenses unless fully hydrated (wet) in HORIEN ULTRA COMFORT.
 - Always keep your lens case clean. Wash & rinse lens case with tap water and air dry every week.
 - Check before insertion that the lens is not turned inside-out.
 - When inserting your lenses always start with the same eye to avoid confusion.
 - The lens can never be lost in your eye. Use a well-lit mirror to help locate your lens, it will probably be on the inside of your eyelid.
 - Do not re-center the lens on the cornea with your finger on the lens. Instead use your eyelid to move and reposition the lens.
 - Avoid touching lenses with your finger nails. Make sure your finger nails are short & smooth.
 - If the lens is nicked or torn do not wear it.
 - Avoid rubbing your eyes after you have inserted lenses. This can irritate the eye or dislodge the lens.
 - Do not allow lotions, soaps, creams, deodorants, hair sprays & cosmetics (especially metallic powders) to come in contact with the lenses. They can cause an eye infection or irritation.
 - Apply make-up only after inserting your lenses.
- NOTE: Avoid eye make-up like eye liners as these can stain your lens.
- Do not wear lenses overnight.
 - Avoid wearing lenses around fumes, irritating vapours, smoking or dusty conditions. This may irritate your eye and/or damage the lens.
 - Always wear swimming goggles while swimming with your lens on.
 - On holiday carry ample supply of HORIEN ULTRA COMFORT.
 - Make sure you have a spare pair of contact lenses.
 - Visit your eye-care practitioner regularly for a routine check-up.

COMPLIANCE AND HYGIENE

- Manufacturer urges customers to follow this recommendation for Clean and Safe Contact Lens wear.
- Carefully and regularly clean contact lenses, as directed by an optometrist / lens care practitioners and the patient information included with lens care products.
- Wash and dry hands before putting on or removing lenses.
- Do not wear lenses beyond the recommended period.
- Store lenses in the proper case and replace the lens storage case at least every three months. Clean the contact lens case after each use with Lens Cleaning Solution and keep the case open and air dry between cleanings.
- Only fresh solution should be used to clean and disinfect contact lenses. Never re-use old solution. Contact lens solution must be changed every day, even if the lenses are not used daily.
- Remove and stop using lenses if you experience unusual eye symptoms; seek medical consultation if necessary.

WASTE DISPOSAL

- Pour unused solution down a drain.
- Dispose of empty bottle in accordance with local waste disposal regulations.

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HORIEN is a registered trademark of Horien
STERICON PHARMA PVT. LTD.
Medical Ltd. / Haichang Contact Lens Co., Ltd.